


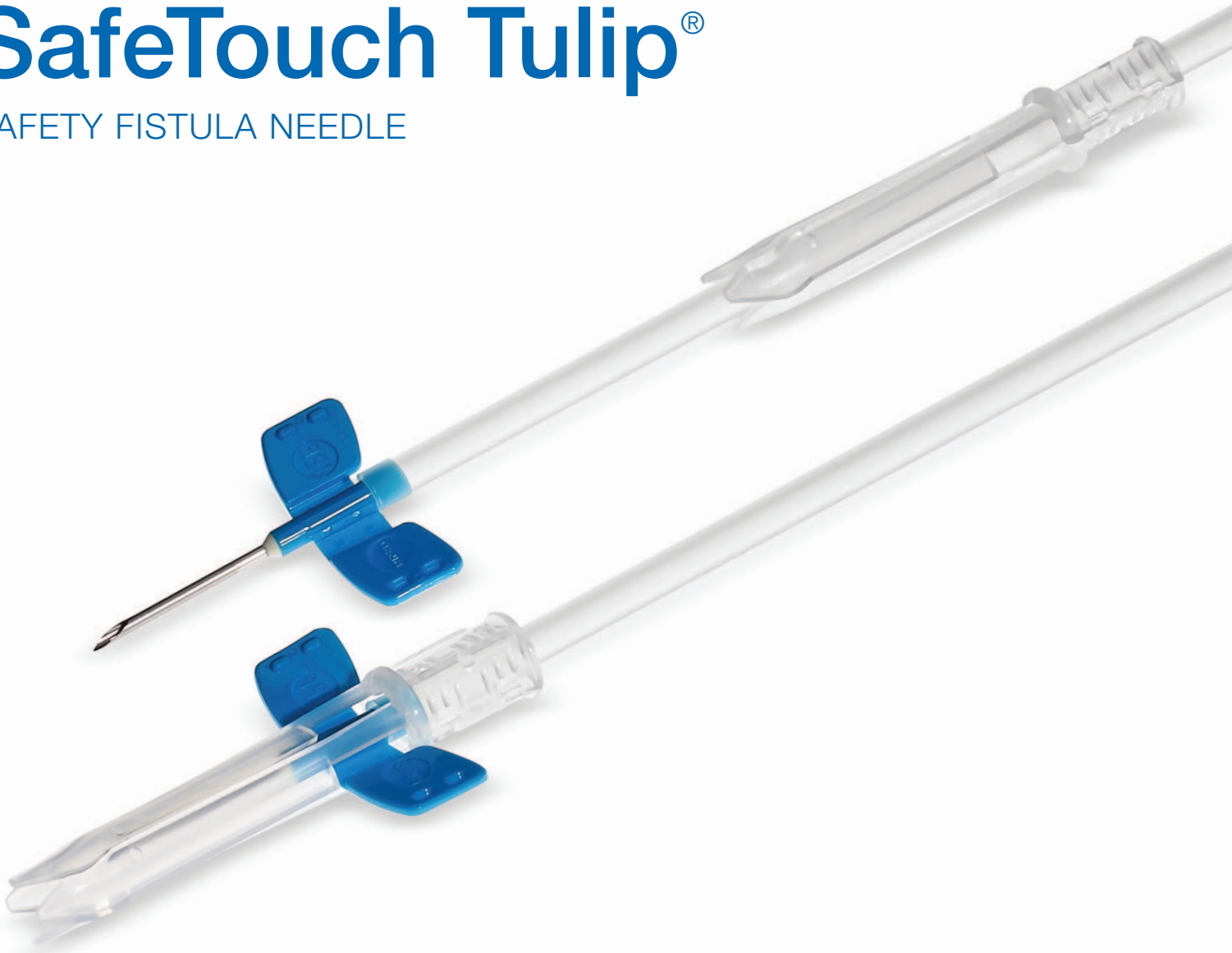

SafeTouch Tulip[®]

SAFETY FISTULA NEEDLE



Features and Benefits of the Tulip Safety Fistula Needle:

- Ease of use during patient cannulation and needle placement
- Audible click indicating safety guard is locked
- Ease of safety activation using a one or two-handed technique



NIPRO CONNECT[®] **PROGRAM**

Our commitment to strengthen every partnership.

All Nipro renal products are backed by the Nipro Connect Educational Program – offering clinicians product evaluations, in-services, continuing education units (CEU's) and instructional materials.

SafeTouch Tulip Needle Removal Technique



1

Remove all securing tape from the wings and the access area. Place a sterile dressing over the needle access site, and bring the safety guard forward towards the wing area.



2

While holding a sterile dressing over the needle access site, place a second finger at the end of the safety guard rigid holding area. Slide the Tulip safety guard forward until 2 petals of the Tulip safety guard tip are positioned over the flattened wings. Note, 2 different finger placement options are shown above.



3

While holding the safety guard firmly, grab and pull back on the tubing until the wings are locked, and the safety guard is fully secured, with an audible and tactile click.



4

Dispose of the SafeTouch Tulip needle in an approved biohazard sharps container.

ORDERING INFORMATION

Item #	Description	Packaging
FT+142530BC	14GX1 Fixed Hub 12" Tubing	50/Box 6 Boxes/Case (300)
FT+152530BC	15GX1 Fixed Hub 12" Tubing	
FT+153230BC	15GX1.1/4 Fixed Hub 12" Tubing	
FT+162530BC	16GX1 Fixed Hub 12" Tubing	
FT+163230BC	16GX1.1/4 Fixed Hub 12" Tubing	
FT+172530BC	17GX1 Fixed Hub 12" Tubing	
FT+173230BC	17GX1.1/4 Fixed Hub 12" Tubing	
FT+152530BCR	15GX1 Rotating Hub 12" Tubing	
FT+153230BCR	15GX1.1/4 Rotating Hub 12" Tubing	
FT+162530BCR	16GX1 Rotating Hub 12" Tubing	
FT+163230BCR	16GX1.1/4 Rotating Hub 12" Tubing	
FT+172530BCR	17GX1 Rotating Hub 12" Tubing	

Item #	Description	Packaging
FT+142530TP	14GX1 Twin Pack 12" Tubing	2/Package: Red - Arterial Blue - Venous
FT+143230TP	14GX1.1/4 Twin Pack 12" Tubing	
FT+152530TP	15GX1 Twin Pack 12" Tubing	
FT+153230TP	15GX1.1/4 Twin Pack 12" Tubing	
FT+162530TP	16GX1 Twin Pack 12" Tubing	
FT+163230TP	16GX1.1/4 Twin Pack 12" Tubing	
FT+172530TP	17GX1 Twin Pack 12" Tubing	
FT+173230TP	17GX1.1/4 Twin Pack 12" Tubing	
FT+142530TPR	14GX1 Twin Pack Rotating Hub 12" Tubing	
FT+152530TPR	15GX1 Twin Pack Rotating Hub 12" Tubing	
FT+162530TPR	16GX1 Twin Pack Rotating Hub 12" Tubing	
FT+172530TPR	17GX1 Twin Pack Rotating Hub 12" Tubing	

CAUTION: Federal law restricts these devices to sale by or on the order of a physician.



NIPRO MEDICAL CORPORATION
200 Crossing Blvd., Bridgewater, NJ 08807
T: +1 (908) 393-7030, F: +1 (908) 393-7031
www.nipro.com

© 2014 Nipro Medical Corp. All Rights Reserved.
SafeTouch Tulip is a registered trademark of Nipro Corporation.
Printed in U.S.A. RN-DSUS-TULIP Rev 0