

Needlestick prevention that meets your needs



- ▶ virtually eliminates exposure
- ▶ single-handed, passive activation
- ▶ hands remain behind the sharp
- ▶ integrated safety mechanism
- ▶ prevents recapping and needle removal
- ▶ remains safe through disposal
- ▶ easy to use, requires minimal training
- ▶ safe and effective for patients

Catalog Number	Description
VanishPoint® U-100 insulin syringes 100 per box/800 per case	
15221	1/2ml VanishPoint insulin syringe 30G x 1/2"
10211	1ml VanishPoint insulin syringe 29G x 1/2"
VanishPoint® syringes 100 per box/800 per case	
10131	1ml VanishPoint tuberculin syringe 27G x 1/2"
10151	1ml VanishPoint tuberculin syringe 25G x 5/8"
10161	1ml VanishPoint tuberculin syringe 25G x 1"
3ml VanishPoint® syringes 100 per box/600 per case	
10301	3ml VanishPoint syringe 25G x 5/8"
10391	3ml VanishPoint syringe 25G x 1"
13011	3ml VanishPoint syringe 25G x 1 1/2"
10311	3ml VanishPoint syringe 23G x 1"
10321	3ml VanishPoint syringe 23G x 1 1/2"
10331	3ml VanishPoint syringe 22G x 1"
10341	3ml VanishPoint syringe 22G x 1 1/2"
10361	3ml VanishPoint syringe 21G x 1 1/2"
5ml VanishPoint® syringes 100 per box/600 per case	
10511	5ml VanishPoint syringe 25G x 1 1/2"
10531	5ml VanishPoint syringe 22G x 1"
10541	5ml VanishPoint syringe 22G x 1 1/2"
10561	5ml VanishPoint syringe 21G x 1 1/2"
10581	5ml VanishPoint syringe 20G x 1 1/2"
10ml VanishPoint® syringes 100 per box/600 per case	
11011	10ml VanishPoint syringe 25G x 1 1/2"
11041	10ml VanishPoint syringe 22G x 1 1/2"
11051	10ml VanishPoint syringe 21G x 1"
11061	10ml VanishPoint syringe 21G x 1 1/2"
11081	10ml VanishPoint syringe 20G x 1 1/2"
VanishPoint® Allergy Syringe Tray 25 per tray/1000 per case	
10134*	1ml VanishPoint allergy syringes 27G x 1/2"

Additional sizes available. For a complete list, please contact RTI or visit our website at www.vanishpoint.com.



© 2008 Retractable Technologies, Inc.
All rights reserved.M005 REV.9

511 Lobo Lane
Little Elm, Texas 75068-0009 USA
Tel.: (972) 221-6644
Fax: (972) 294-4400
Toll Free: 1-888-703-1010
domestic: rtisales@vanishpoint.com
international: rti.intl@vanishpoint.com

UNCOMPROMISING SAFETY FOR CLINICIANS AND PATIENTS



VANISHpoint™
SYRINGES

The New Standard for Safety™

VANISHPOINT® Safety by design

Safety mechanism is built into the syringe

Needle retracts directly from patient into syringe, effectively reducing exposure to contaminated sharps

Attached needle design:

- maintains a closed system
- prevents leakage and contamination
- reduces deadspace
- prevents unnecessary needle changes

"Changing needles between drawing vaccine from a vial and injecting it into a recipient is unnecessary."*

Safe for you and your patients



*CDC (2002). General recommendations on immunization: recommendations of the Advisory Committee on Immunization Practices (ACIP) and the American Academy of Family Physicians (AAFP). MMWR 51(RR02):1-36.

1

Use standard procedure for drawing up medication.



2

Prepare and give injection using aseptic technique according to institutional policy.



3

While the needle is still in the patient, fully depress the plunger to activate retraction.



4

Dispose of VanishPoint® syringe in an appropriate sharps container.



the VANISHPOINT® ADVANTAGE

Designed for Clinician
and Patient Safety

Clear, unobstructed calibrations allow for accurate dosing

Triple beveled, lubricated needle provides patient comfort

EASY, ONE-HANDED ACTIVATION

NO CONTAMINATED SHARP EXPOSED

REDUCED DEADSPACE

Attached needle prevents leakage and contamination

Pre-removal activation prevents exposure to contaminated sharp

NOW AVAILABLE: 1/2ml Insulin Syringes

VanishPoint® syringes are available with a variety of needle sizes and gauges attached. Syringes are color-coded for needle gauge.



Activated VanishPoint® syringes require less disposal space than other syringes and prevent disposal-related injuries.

Other Available VanishPoint® Safety Products

NEW!



VanishPoint® IV Catheter



VanishPoint® Blood Collection Tube Holder

